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- Rudmik L, et al. (2013). Topical therapies in the management of chronic rhinosinusitis: an evidence-based review with recommendations. International Forum of Allergy and Rhinology, 3 (4), 281-298.
- King D, et al. (2015). Saline nasal irrigation for acute upper respiratory tract infections. The Cochrane database of systematic reviews (4).
- Rosenfeld RM, et al. (2007). Clinical practice guideline: adult sinusitis. Otolaryngology - Head and Neck Surgery, 137 (3 Suppl), S1-S31. Rosenfeld RM, et al. (2015). Clinical practice guideline: adult sinusitis update. Otolaryngology - Head and Neck Surgery 152 (2S) S1-S39.
- Gallant JN, et al. (2018). Nasal saline irrigation in pediatric rhinosinusitis: A systematic review. International Journal of Pediatric Otorhinolaryngology, 108, 155-162.
- Slapak I, et al. (2008). Efficacy of isotonic nasal wash (seawater) in the treatment and prevention of rhinitis in children. Archives of otolaryngology - Head and Neck, 134 (1), 67-74.
- Chirico G, et al. (2014). Nasal congestion in infants and children: a literature review on efficacy and safety of non-pharmacological treatments. Minerva Pediatrica, 66 (6), 549-57.

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