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**IMPORTANT PRODUCT USE AND SAFETY INFORMATION. PLEASE READ CAREFULLY AND RETAIN FOR FUTURE REFERENCE. INCORRECT CONTACT LENS CARE CAN CAUSE SERIOUS PROBLEMS WITH THE EYES.** SOLO CARE AQUA® is a one bottle multi-purpose/disinfecting solution that cleans, rinses, disinfects, stores, removes protein and lubricates all soft contact lenses. Isotonic, aqueous anti-microbial/anti-bacterial\* solution containing: polyhexanide 0.0001% (active ingredient and preservative), HydroLock® (dexpanthenol and sorbitol), sodium phosphate, tromethamine, poloxamer 407, disodium edetate.

**SOLO CARE AQUA®** Processed by aseptic techniques. The MicroBlock® antibacterial lens case contains silver.

\*active against eye-pathogen germs listed in ISO 14729

#### INSTRUCTIONS FOR USE

**1. Wash your hands and dry them with a clean towel**

**2. Open the lens case**

**3. Remove lens and place in the palm of your hand**

**4. Apply at least 3 drops of SOLO CARE AQUA® to each lens surface (see picture 1)**

**5. Gently rub the lens using a circular motion for at least 10 seconds**

**6. Rinse the lens thoroughly with SOLO CARE AQUA® while rubbing the lens gently between the thumb and forefinger for at least 10 seconds (see picture 2)**

**7. Place the lens in the lens case and fill one side to the line with fresh SOLO CARE AQUA®**

**8. Replace cap and screw closed**

**9. Repeat procedure with your other lens**

**10. ALLOW YOUR LENSES TO SOAK FOR AT LEAST 5 MINUTES**

**11. Your contact lenses may now be inserted directly into your eyes**

**12. Discard solution from lens case, rinse it with fresh SOLO CARE AQUA® and close until next use**

**13. If the lenses have been stored in SOLO CARE AQUA® for more than 30 days before wearing, you must discard the solution from the case and repeat the manual cleaning process and disinfect the lenses in SOLO CARE AQUA® for at least 5 minutes before wearing them**

**CONTRAINDICATIONS** (reasons not to use)

• Do not use if you are sensitive to any of the ingredients or if you are allergic to silver or

other metals

• Never use SOLO CARE AQUA® for heat disinfection

#### PRECAUTIONS

• Do not use after expiry date

• Check the dropper tip is sealed before first use. If already opened, do not use.

• Close bottle after use

• ONCE BOTTLE IS OPEN, DISCARD ANY REMAINING SOLUTION AFTER 3 MONTHS

• Never touch the dropper tip of the bottle to any surface, since this may contaminate the solution

• Replace lens case with every new bottle of SOLO CARE AQUA®

• Store below 25°C

• Consult your eye care professional before changing solutions

#### WARNINGS

• Always use fresh SOLO CARE AQUA® solution – never re-use solution in your lens case for a second time as efficacy is no longer provided

• NEVER transfer SOLO CARE AQUA® to another container

• NEVER dilute or mix SOLO CARE AQUA® with any solution as this will destroy its performance

• NEVER ALLOW CONTACT LENSES TO COME INTO CONTACT WITH NON-STERILE LIQUIDS (INCLUDING TAP WATER AND SALIVA) AS MICROBIAL CONTAMINATION CAN OCCUR, WHICH MAY LEAD TO PERMANENT EYE DAMAGE

• ALWAYS USE THE SOLO CARE AQUA® SYSTEM COMPONENTS INCLUDING THE LENS CASE

• KEEP OUT OF REACH OF CHILDREN

• Consult your doctor or eye care professional before using any medication with contact lenses

#### ADVERSE REACTIONS

If you experience persistent discomfort or irritation remove your lenses, stop using the product and consult your eye care professional.

**MicroBlock® LENS CASE** The MicroBlock® antibacterial lens case may slightly discolour over time; this discoloration does not affect the anti-microbial properties of the lens case.

SOLO CARE AQUA® is available in 90 ml and 360 ml bottles, including a disposable lens case.

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**© 2016, Menicon Co., Ltd.**
SOLOCARE AQUA, MicroBlock, HydroLock are registered trademarks of Novartis AG.
**Menicon Co., Ltd.**
**Menicon Pharma SAS**