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Made in France

Laboratoires URGO HEALTHCARE - 42 rue de Longvic - 21300 Chenôve - France

EN Self-adhesive restraining bandage

DESCRIPTION

Length extensible woven bandage coated by thin latex particles.
Cotton, viscose, elastane.

PROPERTIES

- Cohesive: the bandage adheres only to itself and not to the skin or hair.
- Comfort: is well aerated, thus increasing patient comfort.
- Extension greater than 120%.
- Restraining force 1: 20 to 45cN/cm at 30% extension.
- Easy to use: can be cut without using scissors.
- Can be repositioned.
- Water-resistant: the bandage can be washed once or twice.
- Radiotransparent.

INDICATIONS

Bandage for restraining and keeping dressings in place.

Traumatology:

- In the prevention of accidents such as sprains, luxations and subluxations.
- In the treatment of injuries such as pulled and strained muscles.
- To protect friction zones.

Phlebology:

- Prevention and treatment of essential varicose veins or of those following phlebitis, varicose ulcers, and acute or subacute hypodermic inflammation.
- Reduction and stabilization of oedema of venous origin, and of lymphoedemas.
- Prevention of thrombo-embolisms and thrombophlebitis.
- Restraining after stripping or sclerosing treatment.

Nursing care:

- Dressing maintains.

INSTRUCTIONS FOR USE

1. Unroll 20 to 30 cm so as to surround the part of the body that is to be protected, and relax the tension on the bandage.
2. Make a first turn with the bandage, and then continue bandaging, covering each turn with at least half of the next turn.
3. To ensure a good fit, the last turn must not touch the skin but must cover the previous turn completely.
4. Finish bandaging by pressing the end of the bandage with palm of the hand for a few seconds.
5. The last turn with the bandage must be applied WITHOUT ANY TENSION.
6. Upkeep :
 - Wash by hand or in a washing machine in water 40°C maximum, without detergent or chlorine.
 - Rinse, lay bandage out flat and leave to dry.

WARNINGS AND PRECAUTIONS

- Never apply directly on an injured skin.
- In the case of dressing fixing, it is advisable to sterilize the bandage.
- When applied, the embossing has to stay visible, this guarantees a secure application.
- If the bandage is too tight reduce the tension.
- In phlebology, the bandage has to be put before getting up - from toes, by pulling up along the leg - and kept until the bedtime.
- In case of prolonged or repeated use, the elastic thread can be weakened by the use of fatty compounds under the bandage.

CONTRA-INDICATIONS

- Arterial conditions (arterial or predominantly arterial ulcers; known or suspected arterial disease).
- Ankle Brachial Pressure Index (ABPI) < 0.8.
- Patients suffering from diabetic microangiopathy, ischaemic phlebitis (deep vein thrombosis), septic thrombosis.
- Ulceration due to infection.
- Ulceration to latex or any of the components.
- Stadium sprains stages III and IV.

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